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DESINFEKATIOVALAISIN

BT 90S Jalkamalli (FM)

BT 90S Seinä-/kattomalli (CM/WM)

Operating, Maintenance and Installation Manual

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Read this before beginning to use the disinfection lamp

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Contents

- 1 Technical Specifications
- 2 Features
- 3 Installation
- 4 Operating
- 5 Maintenance
- 6 Spare parts
- 7 Wiring diagrams

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1. BT 90S Technical Specifications

Radiator	3 x 30 W UVC-radiators
Fan	9 W
UV-radiation	254 nm
Measured cleaning capacity	23m ³ /h
Cleaning effectiveness	95 – 98 % of bacteria 93 – 96 % of fungi
Service	once a year
Sound volume	33/35 dB (A)
Total power consumption	110 W
Weight	11 kg (CM/WM) 24 kg (FM)
Dimensions	1400 x 300 x 150 mm (CM/WM) 1700 x 660 x 630 mm (FM)

SYMBOLS:

Misuse may result a faulty operation of the disinfection lamp.

Misuse may result a damage to the lamp. UVC rays are dangerous to the skin and eyes. See further instructions in the manual.

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
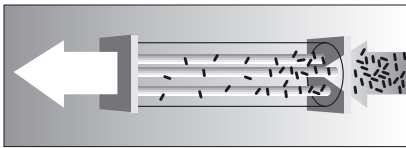
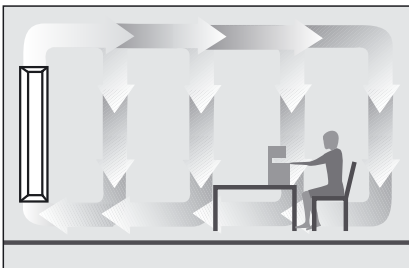
2 Features


Please read this manual carefully before installing and connecting the disinfection lamp. When the lamp is operational there are UVC rays inside the disinfection lamp. The device **MUST NOT** be opened when it is connected to the mains

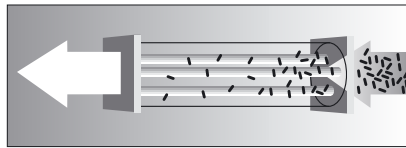- BT90 S / BT90S FM clear effectively the air from bacteria and fungi
- Disinfection lamp is suitable for air purification in the whole health care sector where hygienic conditions are needed, for instance laboratories, operating departments, isolated departments and catering in hospitals.
- Cleaning is possible 24 hours a day
- Disinfection lamp can be installed to the wall or ceiling. The floor model is equipped with castors and can be moved around as needed.
- A fan circulates the air through UVC radiation sources. The UVC radiation is kept inside the disinfection lamp cover

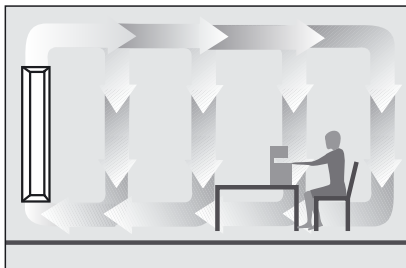- The personnel working in the facilities will not be exposed to harmful UVC radiation, because the cleaning chamber is isolated with reflecting surfaces.

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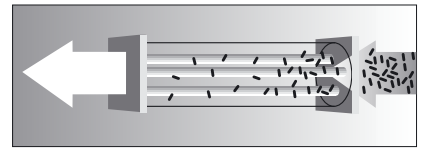
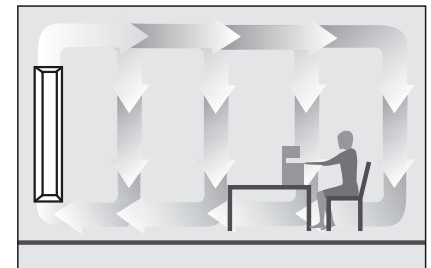
General

FLOOR MODEL (FM)

BT90S FM (floor model) equipped with castors (4 pcs) is working when the power cord is connected to the mains.

CEILING AND WALL MODEL (CM/WM)

The lamp is working when the power cord is connected to the mains. The lamp must be installed as specified in the instructions. Make sure the lamp is installed in an appropriate environment, i.e. the lamp must not become subject to mechanical stress, shocks or tampering. Furthermore, note the markings in the type plate.

Wall installation (WM)

The disinfection lamp must be mounted on the wall at right angles to the floor. Place it with the narrow side upwards in a way that the electrical connector faces the ceiling. The recommended installation height from the floor is 60 cm measured from the floor to the bottom of the lamp cover. The normal direction of the airflow is from the bottom upwards, in which case the fan is in the top of the lamp. If necessary, the airflow direction can be turned by turning the fan. The power can be switched on or off by connecting or disconnecting the power cord from mains.

Ceiling installation (CM)

The disinfection lamp is mounted on the ceiling with its back facing the ceiling. The minimum distance from walls or other obstacles is 60 cm. If necessary, the airflow direction can be turned by turning the fan. If the room is low (less than 2.5 metres), diffuse reflectance must be taken into account. In narrow spaces the lamp must always be mounted transversely to the space. In high and wide spaces the direction can be determined freely. The power can be switched on or off by connecting or disconnecting the power cord from mains.

Make sure the disinfection lamp is mounted safely on the ceiling.

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4 Operation

Applications

The disinfection lamp is intended for premises with special air purity requirements. These include:

- patient areas in hospital
- instrument maintenance areas in hospital
- operating theatres
- laboratories
- outpatient areas
- kitchens or catering in hospital
- waste disposal areas
- areas with odour problems
- areas with excessive airborne microbes
- areas into which the entry of microbes needs to be prevented

Determination of application space size

As a general rule, the air change rate should be approximately five times the total volume of air in the room over 24 hours. The number of devices required depends on how demanding the space is:

- amount of air impurities
- environmental conditions
- desired level of hygiene

In normal conditions the purification capacity of one device is approximately 110 m³. In demanding conditions the purification capacity may be only one tenth of this.

Example of calculation: how many devices for required space:

Device's measured power is 6,35 liters per second
Amount of circulated air /24 hours is 550m³
Cleaning capacity of device in normal circumstances is 550 m³ : 5 (air change rate) = 110 m³

For example:

Size of operating room
4.4m x 5.8 m x 3.4 m = 87m³
Circulation speed of cleaned air with one lamp = 550 : 87 = 6.3 times/24h, with two lamps 12.6 times/24h and with four lamps 25.2 times/24h.

The most reliable way to assess the purification capacity of the device is to measure the space both before and after the installation of the device.

Required level of air circulation:

Corridors and waiting rooms
air change rate 5
Operating rooms
air change rate 25
Biowaste rooms and rooms with very demanding circumstances
air change rate 60

Application environment

The instructions on the type plate must be complied with. For example, IP 20.

Operation

The device is connected to the mains, after which the device operates automatically.

Ensuring the functioning of the device

The functioning of the device is best ensured through regular maintenance. The maintenance instructions describe the necessary measures. The effect of the UVC lamps remains optimal during the period of time specified by the manufacturer, after which the lamps must be replaced.

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5 Maintenance

Maintenance instructions

The device must be serviced by an authorised service shop only. Because there are UVC lamps inside the device, the device must ALWAYS be totally disconnected from the mains before servicing. Only original components can be used as spare parts. Any damaged parts must always be replaced.

The servicing interval for the disinfection lamp is 12 months. The following components MUST be replaced during the annual servicing:

- 1) UVC tubes
- 2) tube holders
- 3) tube starters

In addition, the faultlessness of all the other components must be ensured. UVC radiation embrittles plastic parts.

Cleaning

- The disinfection lamp must be disconnected from the mains during cleaning
- Clean the outer surfaces of the device using a cloth dampened with an alkaline cleaning solution. The cleaning solution must not contain abrasive substances
- Make sure no cleaning solution enters the disinfection lamp

If necessary, the UVC exposure chamber of the device can be cleaned using lint-free cloth. NO cleaning solution must be used. The exposure chamber of the device is, if necessary, cleaned during annual servicing, and this can only be carried out by an authorised service shop.

Life of the UVC lamps

The effective life is as specified in the instructions provided by the manufacturer. The UVC lamps must be replaced every 12 months during the annual servicing.

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